Medtronic

MIDAS REX[®] LEGEND[®]

Pneumatic High Speed System

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	Attention: See Instructions for use
	For U.S. audience only Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician
	Catalog Number, Reference Number, Reorder Number, Part Number
	Lot Number
	Serial Number
	Quantity
	Single use only; do not reuse
	Is approximately equal to
	Non-sterile
	Sterilized by gamma irradiation
	Use by date
	Date of Manufacture
	Storage Temperatures
	Unlock
	Lock
	On
	Off
	Finger-Operated Control
	Foot-Operated Control
	Conformité Européene (European Conformity) This symbol means that the device fully complies with European Council Directive MDD 93/42/EEC.
	Authorized Representative in the European Community
	Manufacturer
	DO NOT Dispose to unsorted municipal waste
TOOL	Tool Control
TUBE	Tube Control

GENERAL OPERATING GUIDELINES	2
WARNING SUMMARY	3
INTRODUCTION — Legend® System Components	7
Legend Motors	7
Non-Disposable Legend Accessories	8
Legend Pneumatic Control Unit	8
Legend Regulator	8
Legend Instrument Cases	8
Legend Adapters	9
Legend Cleaning Nozzle	9
Legend Attachments	10
Disposable Legend Accessories	11
Legend Lubricant/Diffuser	11
Cleaning Brushes	11
Legend Dissecting Tools	12
General Guidelines for Attachment and Tool Applications	14
Operating Room Set Up	16
Power Source Requirements	16
System Connections	16
PRE-OPERATIVE PROCEDURES	18
LEGEND MOTOR FEATURES	22
POST-OPERATIVE PROCEDURES	23
Disassembly of Equipment	23
Cleaning	24
Sterilization Guidelines for Legend Motors and Attachments	27
LEGEND ATTACHMENTS AND DISSECTING TOOLS	28
Straight Attachments	28
Angled Attachments	29
Footed Attachments	30
Contra-Angle Attachment	31
Metal Cutting Attachment	32
Telescoping Attachment	33
Perforator Attachment	34
Jacobs Chuck Attachment	35
TECHNICAL INFORMATION — Legend System Troubleshooting	36
MIDAS REX® LEGEND® SERVICE AND WARRANTY	42

The MEDTRONIC MIDAS REX® LEGEND® Pneumatic High Speed System ("LEGEND SYSTEM") is intended for use in surgical procedures for the following medical applications:

- Neurosurgical
- Spine
- Ear, Nose and Throat (ENT)
- Orthopaedic Surgery
- General Surgery and Plastic Surgery including Maxillofacial, Craniofacial and Sternotomy

Contraindications: None

The LEGEND System is comprised of the following:

- Legend Gold, Legend Gold Touch or Legend Platinum Motors
- Legend Pneumatic Control Unit with Regulator and various connectors
- Legend Attachments
- Legend Dissecting Tools — single use, disposable
- Legend Lubricant/Diffuser — single use, disposable

Read and understand this instruction manual before use of the LEGEND SYSTEM.

The LEGEND SYSTEM is designed for use by medical professionals familiar with powered surgical instrumentation. The surgeon is responsible for learning the proper techniques in the use of this system, as inappropriate use may potentially be harmful. It is strongly recommended that the surgeon and dedicated operating room personnel are knowledgeable with the use of this equipment by being trained in Medtronic Midas Rex Hands-On Workshops or by one of the local authorized representatives.

If Legend components require servicing or refurbishing, return components to Medtronic Powered Surgical Solutions (MPSS) for quality assured service by factory-trained personnel who will utilize genuine MEDTRONIC MIDAS REX® LEGEND® parts as required. All Legend System components returned for servicing or refurbishing should be properly cleaned and sterilized prior to shipping.

! USA Rx only Federal Law (USA) restricts this device to sale by or on the order of a physician.

For Questions or Problems, consult the Legend Instruction Manual, or contact your Medtronic Neurologic Technologies sales representative, or call:

- Medtronic Powered Surgical Solutions (MPSS) Technical Service:
- 8 a.m. to 6 p.m. (Central Standard Time)
- Telephone: 800-643-2773 or 817-788-6400

Outside the U.S.A., contact your nearest Medtronic Regional Distributor or Medtronic Neurological Sales Representative or call U.S.A. Telephone 817-788-6400

 This symbol is used to alert the reader to important safety and precautionary information. When displayed on the actual device or packaging, it refers the user to the instruction manual.

The words Warning, Caution and Note carry special meanings and should be carefully reviewed.

 **WARNING** The personal safety of the patient, operator and/or operating room staff may be involved. Disregarding this information could result in injury to the patient, operator and/or operating room staff.

 **CAUTION** These instructions point out special service procedures or precautions that must be followed to avoid damaging the instrument. A damaged instrument could result in injury to the patient, operator, and/or operating room staff.

NOTE This provides special information to make maintenance easier or important operating instructions more clear.

Read and understand warnings before use of the LEGEND SYSTEM.

Important Safety Instructions:

 **WARNING**

General:

DO NOT use any other components except for Medtronic Midas Rex Legend Attachments, Legend Dissecting Tools and Legend accessories in combination with a Legend Motor. Use of devices other than Legend Systems or associated parts or components may cause injury to the patient, operator and/or operating room staff. Use of devices other than Legend Systems or associated parts or components will damage the motor and attachments and will void manufacturer's limited warranty.

DO NOT interchange components of the Legend System with components from the Medtronic Midas Rex® Classic Motor System, the Midas Rex® III Motor System or the Mednext® Bone Dissecting System and their associated attachments and dissecting tools.

DO NOT operate Legend System in a Magnetic Resonance Imaging Suite.

DO NOT use a Legend Motor and Legend Attachments before proper cleaning and sterilization protocol is followed.

DO NOT dissect bone without adequate irrigation to prevent thermal necrosis.

DO NOT operate the Legend Motor without eyewear protection.

DO NOT use Legend System components if damage is apparent or if components do not run properly. The Legend System must be inspected for damage prior to each use.

- Conduct a visual inspection of the motor's exhaust hose for cracks or tears.
- Check attachments for proper appearance. Install attachment and dissecting tool, then briefly run motor.
- Check motor for overheating and leaking lubricant at motor collet.
- Check attachment for overheating.
- Check dissecting tool for flail.

DO NOT attempt to use an overheating motor or attachment.

For metal transection, observe the following safety precautions:

- a. Eyewear protection is essential.
- b. Irrigate copiously to cool the cutting surfaces.
- c. Protect the wound site from metal debris.
- d. Use a clamp or grasping device to control loose fragments during transection of any metal component.

Legend Motors or Legend Attachments which fail due to extended use may as a result of such failing allow a component to detach and fall from the motor or attachment and may cause patient injury.

Legend Motors:

Heavy side loads may cause overheating of the motor to the point where the motor is uncomfortable to hold.

- Never place an overheated motor on the patient or patient draping during surgery.
- Mitigate the overheating by discontinuing use and rest the motor by using intermittently, or wrap the motor/attachment interface with a moist sterile towel.
- If the motor is passed off, the receiver must grasp the motor by the proximal end close to the motor hose.

DO NOT attempt to use a motor that is leaking lubricant. Legend Motors which fail due to extended use and/or lubricant seal failure may allow foreign matter to migrate or emit from the tool collet end of the motor. Non-sterile fluid may leak into the surgical site. As a result, measures may be required to protect the patient from infection, per the physician's discretion. If a Legend Motor requires servicing or refurbishing, return motor to MPSS.

DO NOT attempt to change a dissecting tool or attachment while the motor is running, or when the motor or attachment is in an overheated state.

DO NOT use a Legend motor if the motor continues to run after releasing the foot pedal or finger control. If a Legend Gold Touch Motor or Legend Pneumatic Control Unit requires servicing or refurbishing, return to MPSS to be refurbished.

The Legend Motor's exhaust hose may have an oily film on the external surface from pressure and/or temperature differentials following sterilization. Wipe exhaust hose with a sterile cloth prior to use. After cleaning, if motor continues to have oil on the exhaust hose, return the motor to MPSS to refurbish.

DO NOT place motor, attachment and dissecting tool on the patient or in an unsecured location during surgery.

DO NOT use Legend Pneumatic Control Unit finger/foot control in the “” position to operate a Legend Gold or Legend Platinum Motor. The control is used in the “” position only for the Legend Gold Touch Motor.

 **DO NOT** install or remove an attachment and dissecting tool from the Legend Gold Touch unless the safety slide is in the “○” position.

 **DO NOT** remove Legend Gold Touch Motor from Pneumatic Control Unit unless compressed gas has been released from the hoses.

Legend Attachments:

DO NOT use a dissecting tool without the appropriate attachment as injury may occur to patient and/or operator.

DO NOT use excessive pressure, such as bending or prying, on attachments or dissecting tools. This may cause tool to bend or break and cause injury to patient, operator and/or operating room staff.

DO NOT use a Legend Attachment if any part of the attachment appears to be bent, loose, missing or damaged. If a Legend Attachment requires servicing or refurbishing, return attachment to MPSS.

DO NOT force or modify any attachment or dissecting tool to fit a Legend Motor.

Heavy side loads and/or long operating periods may cause overheating of the attachment to the point where the attachment is uncomfortable to hold.

- Never place an overheated attachment on the patient or patient draping during surgery.
- Mitigate the overheating by discontinuing use and rest the attachment by using intermittently, or wrap the attachment interface with a moist sterile towel.
- If the attachment is passed off, the receiver of the attachment must handle cautiously.

Remove Legend Footed Attachments cautiously and slowly per instructions to avoid injury to the operator.

Legend Dissecting Tools:

DO NOT use a dissecting tool without the appropriate attachment as injury may occur to patient, operator and/or operating room staff.

DO NOT use or re-process a dissecting tool that has already been used. Using a dissecting tool that has already been used may cause injury to the patient, operator and/or operating room staff. Using a dissecting tool that has already been used may affect motor performance and will void manufacturer's limited warranty.

DO NOT use excessive pressure, such as bending or prying, on attachments or dissecting tools. This may cause tool to bend or break and cause injury to patient, operator and/or operating room staff.

DO NOT use an attachment and dissecting tool combination if flail or vibration occurs.

DO NOT use metal cutting dissecting tools on bone.

DO NOT use a twist drill or contra-angle dissecting tool at an operating pressure over 80 psi (5.5 bar).

DO NOT use a dissecting tool if the sterile packaging is damaged.

If a dissecting tool package is opened, but the tool is not used or contaminated, the tool can be re-sterilized by steam sterilization. Remove tool from all original packaging and place into an approved autoclave package. Steam sterilize as follows:

High Vacuum Steam	270° F (132° C) for 5 minutes
Gravity Displacement	270° F (132° C) for 15 minutes

The re-sterilized tool must be used promptly following re-sterilization. If rust or corrosion is encountered after re-sterilization, do not use the re-sterilized tool.

Dissecting tool flutes are sharp and can perforate surgical gloves. Always handle with care. Do not spin, twist, or pull a tool by grasping the flutes. Gently pull on dissecting tool shaft to ensure that it is fully seated and properly installed in motor.

Legend Pneumatic Control Unit:

DO NOT use the Legend Pneumatic Control Unit to operate any other systems except for Legend Motors and TRITON® Power Surgical Instrument Systems.

DO NOT disassemble equipment before gas pressure is released from Legend Pneumatic Control Unit and associated hoses.

DO NOT carry Legend Pneumatic Control Unit by the N₂ DISS pressure hose.

Legend Lubricant/Diffuser:

Failure to properly secure Legend Lubricant/Diffuser may cause injury to operator and/or operating room staff.


During a lengthy surgical procedure with extended motor use, lubricant may drip from lubricant/diffuser.

LEGEND SYSTEM COMPONENTS

LEGEND MOTORS:

LEGEND GOLD

High speed, high torque pneumatic motor used to dissect bone and biomaterials.

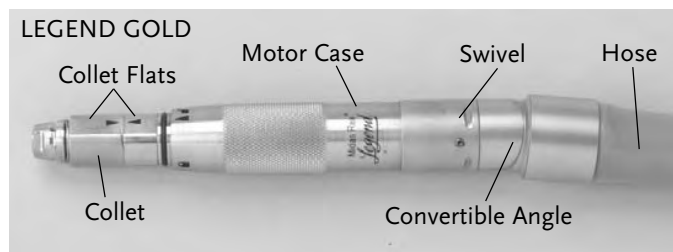
LEGEND GOLD TOUCH

High speed, high torque pneumatic motor with a variable speed finger control. This motor may be used to dissect bone and biomaterials.


Instruction Manual will use “” symbol to indicate information on Legend Gold Touch.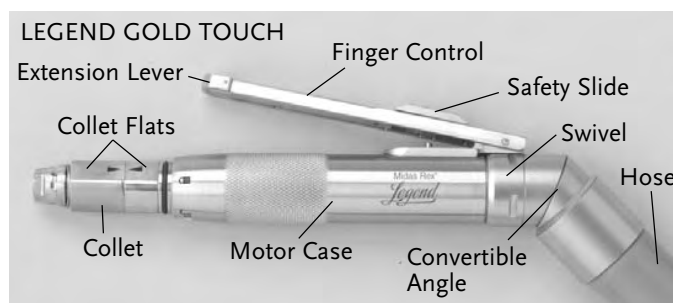

LEGEND PLATINUM

High speed, compact pneumatic motor, with a reduced sound output and slightly decreased power when compared to the Legend Gold and Gold Touch. This motor may be used to dissect bone and biomaterials.

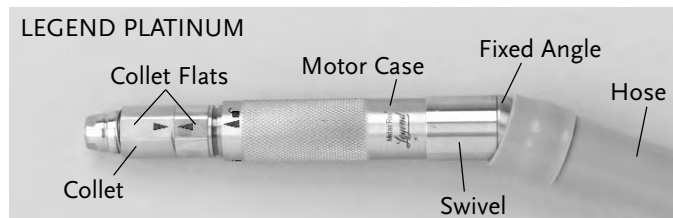NOTE

Each Legend Motor has a Lubricant/Diffuser housing at the end of the motor hose.

Legend motors and associated hoses do not contain latex.

WARNING

DO NOT use any other components except for Medtronic Midas Rex® Legend Attachments, Legend Dissecting Tools and Legend accessories in combination with a Legend Motor. Use of devices other than Legend Systems or associated parts or components may cause injury to the patient, operator and/or operating room staff. Use of devices other than Legend Systems or associated parts or components will damage the motor and attachments and will void manufacturer's limited warranty.

Lubricant/Diffuser Housing

NON-DISPOSABLE LEGEND ACCESSORIES:

LEGEND PNEUMATIC CONTROL UNIT

Controls speed of Legend Gold and Platinum Motors with a variable speed foot pedal, provides finger/foot control for operation of the Gold Touch Motor, supplies port for Legend motors and port for Triton® Power Surgical Instrument System. N₂ DISS pressure hose is 20' long. A longer Medtronic Midas Rex N₂ DISS pressure hose is available and may be changed on-site.

NOTE

Legend Pneumatic Control Unit is provided with a certified Supply Pressure gauge.

Legend Pneumatic Control Unit has a handle for ease of re-positioning unit while on floor or for carrying unit. A hole is provided in the base for hanging unit on a wall hook. Port plugs are attached to prevent debris ingress when a port is not in use.

CAUTION





- Use Legend Pneumatic Control Unit to operate only the Legend Motors and/or TRITON® Power Surgical Instrument Systems.
- Legend Pneumatic Control Unit and pressure hose should not be sterilized.

LEGEND REGULATOR

Controls the delivery pressure of compressed gas to the Legend Pneumatic Control Unit. The pressure gauges monitor cylinder pressure (right gauge) and delivery pressure (left gauge).

LEGEND INSTRUMENT CASES

Used to organize, customize and contain Legend equipment. Available in several sizes with individual labeled tray inserts.

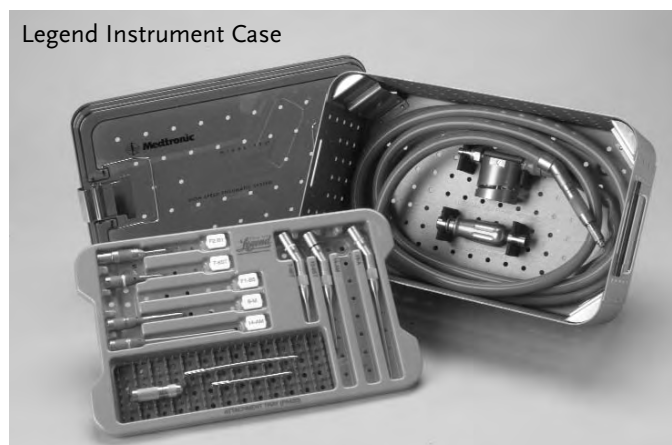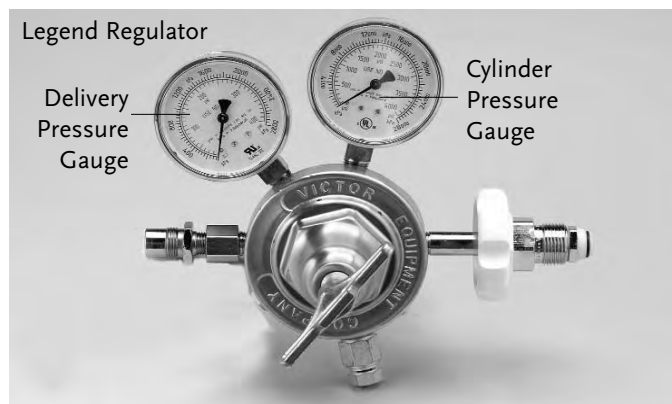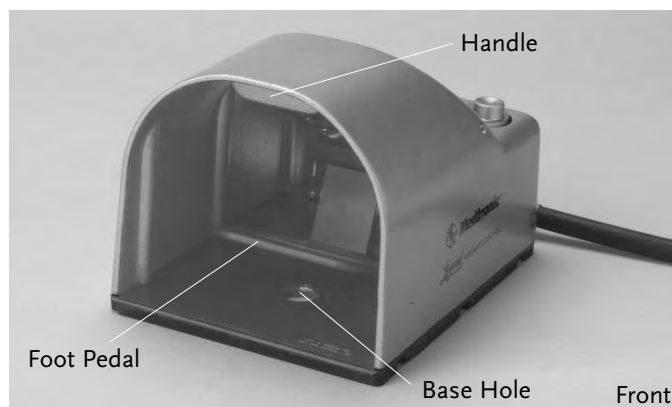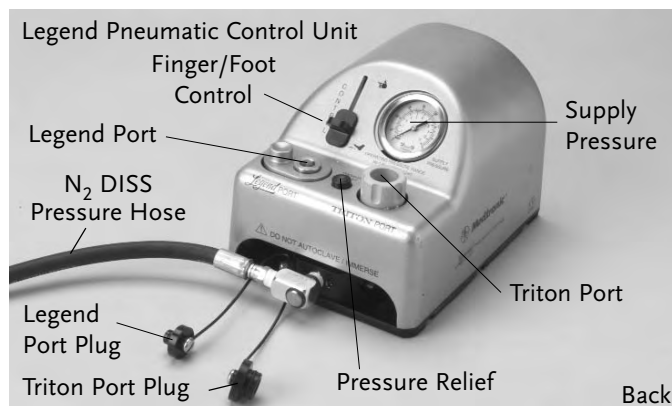N₂ DISS to Male Schrader Adapter

NON-DISPOSABLE LEGEND ACCESSORIES *(continued)*

Legend Adapters:

N₂ DISS to MALE Schrader Adapter

Allows for the Pneumatic Control Unit's N₂ DISS pressure hose connection to be attached to a Female Schrader in-house gas connection.

A N₂ DISS to Female Schrader Adapter is also available for connection of the Pneumatic Control Unit's N₂ DISS pressure hose to a Male Schrader in-house gas connection.


N₂ DISS to Air DISS Adapter

N₂ DISS TO AIR DISS ADAPTER

Allows for the Pneumatic Control Unit's N₂ DISS pressure hose connection to be attached to an Air DISS in-house gas connection.

N₂ DISS to WF4 Adapter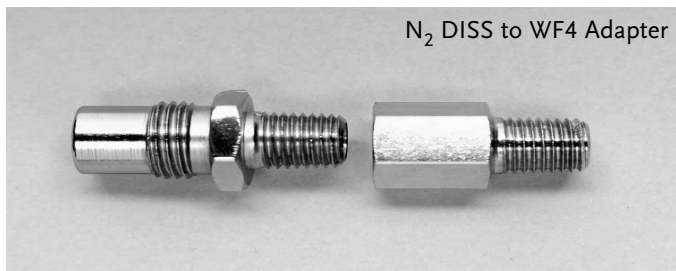

N₂ DISS TO WF4 ADAPTER

Allows for the Pneumatic Control Unit's N₂ DISS pressure hose connection to be attached to a Midas Rex Safety Valve Regulator previously used for Midas Rex Classic or Midas Rex III Motors. The in-line oiler must be removed from the Safety Valve Regulator.


Legend Attachment Cleaning Nozzle

LEGEND ATTACHMENT CLEANING NOZZLE

Allows for recommended aerosol spray cleaner to be used in cleaning procedures.

Legend Motor Wrench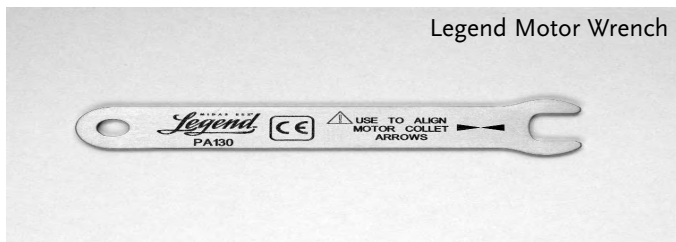

LEGEND MOTOR WRENCH

Used to align arrows on motor collet flats prior to installation of a Legend attachment.

NON-DISPOSABLE LEGEND ACCESSORIES (continued)

LEGEND ATTACHMENTS — Provides support and stability to the rotating dissecting tool.

LEGEND ATTACHMENTS	REORDER NUMBER	MARKING	APPROX. LENGTH	INTERNAL DIAMETER	COLOR BAND
STRAIGHT ATTACHMENTS	AS07	7-6ST	7 cm	2.4 mm	Yellow
	AS08	8-B	8 cm	2.4 mm	Red
	AS09	9-M	9 cm	3.2 mm	Blue
	AS10	10-9ST	10 cm	2.4 mm	Purple
	AS14	14-AM	14 cm	3.2 mm	Green
	AS15	15-A	15 cm	2.4 mm	Orange
	AS21	21-TU	21 cm	3.2 mm	Ochre
	AS26	26-R	26 cm	3.2 mm	Gray
ANGLED ATTACHMENTS	AA07	7-6ST	7 cm	2.4 mm	Yellow
	AA10	10-9ST	10 cm	2.4 mm	Purple
	AA14	14-AM	14 cm	3.2 mm	Green
	AA15	15-A	15 cm	2.4 mm	Orange
FOOTED ATTACHMENTS	AF01	F1-B5	9 cm	2.4 mm	Light Green
	AF02	F2-B1	10 cm	2.4 mm	Light Blue
	AF03	F3-S	12 cm	3.2 mm	White
TELESCOPING ATTACHMENT					
TELESCOPING ATTACHMENT BASE	AT10	—	—	—	—
TUBE – STRAIGHT	TT9	T9	9 cm	1.2 mm	Teal
TUBE – CURVED	TT9C	T9	9 cm	1.2 mm	Teal
TUBE – STRAIGHT	TT12	T12	12 cm	1.2 mm	Light Purple
TUBE – CURVED	TT12C	T12	12 cm	1.2 mm	Light Purple
TUBE – STRAIGHT	TT15	T15	15 cm	1.5 mm	Brown
TUBE – STRAIGHT	TT17	T17	17 cm	1.5 mm	Navy Blue
CONTRA-ANGLE ATTACHMENT	AC16	16-MF	16 cm	3.2 mm	Pink
METAL CUTTING ATTACHMENT	ASMC	MC	9 cm	3.2 mm	Peach
PERFORATOR ATTACHMENT - 800 rpm	AD01	Perforator Driver	13 cm	N/A	None
5/32" JACOBS CHUCK ATTACHMENT	AD02	Jacobs Chuck	12 cm	N/A	None
PERFORATOR ATTACHMENT - 1000 rpm	AD03	Perforator Driver	13 cm	N/A	None

WARNING

DO NOT use any other components except for Medtronic Midas Rex Legend Attachments, Legend Dissecting Tools and Legend accessories in combination with a Legend Motor. Use of devices other than Legend Systems or associated parts or components may cause injury to the patient, operator and/or operating room staff. Use of devices other than Legend Systems or associated parts or components will damage the motor and attachments and will void manufacturer's limited warranty.

NOTE

Angled and straight attachments with the same length, marking and color band share the same dissecting tool. Curved and straight telescoping tubes with the same length, marking and color band share the same dissecting tool. Example: The 14-AM straight and 14-AM angled attachments are 14 cm long, marked 14-AM and have a green color band. All dissecting tools with the prefix 14 (14MH30) may be used in either the 14-AM straight or 14-AM angled attachment.

Color band on attachment is for visual reference. Match the nomenclature on the attachment with the nomenclature on the dissecting tool packaging.

DISPOSABLE LEGEND ACCESSORIES


LEGEND LUBRICANT/DIFFUSER

Provides lubrication to Legend Motors, diffuses spent air from motor and filters oil from exhausted air.


CAUTION

- **DO NOT** exceed 1 hour drill time per surgery for a Legend Lubricant/Diffuser.
- **DO NOT** re-use a used Legend Lubricant/Diffuser.
② Legend Lubricant/Diffuser is a single use, disposable product.
- **DO NOT** steam or EO sterilize the Legend Lubricant/Diffuser. The lubricant/diffuser will melt into the lubricant/diffuser housing with potential damage to the motor. The lubricant/diffuser must be removed from the motor's lubricant/diffuser housing prior to sterilization.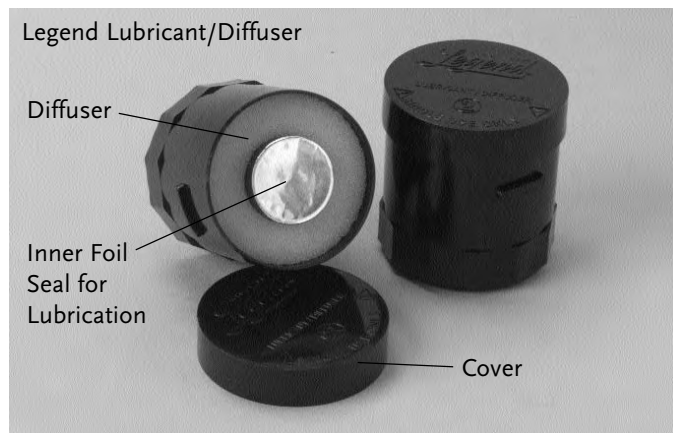


TELESCOPING TUBES

Provides support to the rotating dissecting tool. Telescoping tubes are disposable following multiple uses and should be discarded when heat or excessive vibration is noticed or insertion of tools becomes difficult.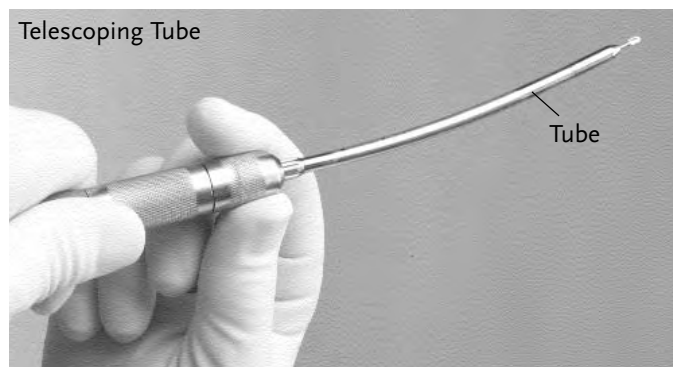

CLEANING BRUSHES

Used to clean debris from lumen of attachments and telescoping tubes. Sized for an internal bore diameter of 3.2 mm, 2.4 mm or 1.2 mm in Legend Attachments and Telescoping Tubes.

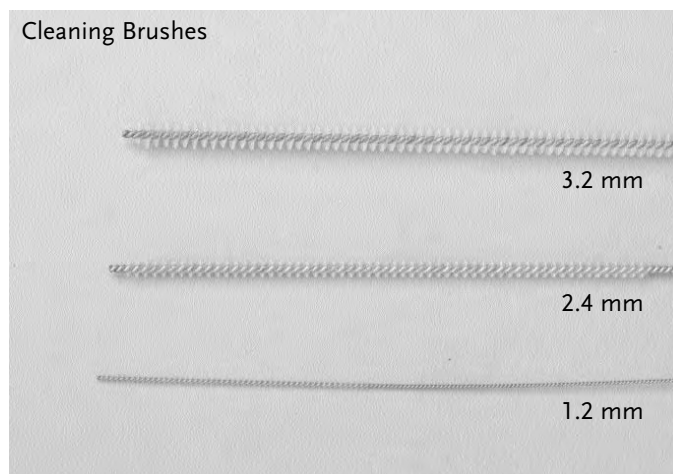DISPOSABLE LEGEND ACCESSORIES (continued)

LEGEND DISSECTING TOOLS

Legend Dissecting Tools' intended use is for cutting bone and biomaterials.

Legend Dissecting Tools are provided as sterile.

NOTE

Dissecting tool's size and geometry may contribute to excessive vibration at certain speeds. Increase or decrease speed by changing pressure setting or foot/finger control. Change to a different dissecting tool.

Dissecting tools with "L" identification are longer tools intended for light bone dissection. The increased dissecting tool stem/head configuration may affect dissection stability.

Surgeons should familiarize themselves with the performance of the dissecting tools before use.

CAUTION

- **DO NOT** use any other dissecting tool other than Legend Dissecting Tools with a Legend Motor.
- Match the nomenclature and color code on the Legend Dissecting Tool packaging to the same nomenclature and color code on the Legend Attachment.
- Use only Legend Dissecting Tools with a Legend Motor.
- ② Legend Dissecting tools are single use devices.

WARNING

DO NOT use or re-process a dissecting tool that has already been used. Using a dissecting tool that has already been used may cause injury to the patient, operator and/or operating room staff. Using a dissecting tool that has already been used may affect motor performance and will void manufacturer's limited warranty.


DO NOT use an attachment and dissecting tool combination if tool flail or excessive vibration occurs.

DO NOT use a twist drill or contra-angle dissecting tool at an operating pressure over 80 psi (5.5 bar).

NOTE

Match the nomenclature and color code on the Legend Dissecting Tool packaging to the same nomenclature and color code on the Legend Attachment.

LEGEND DISSECTING TOOL NOMENCLATURE: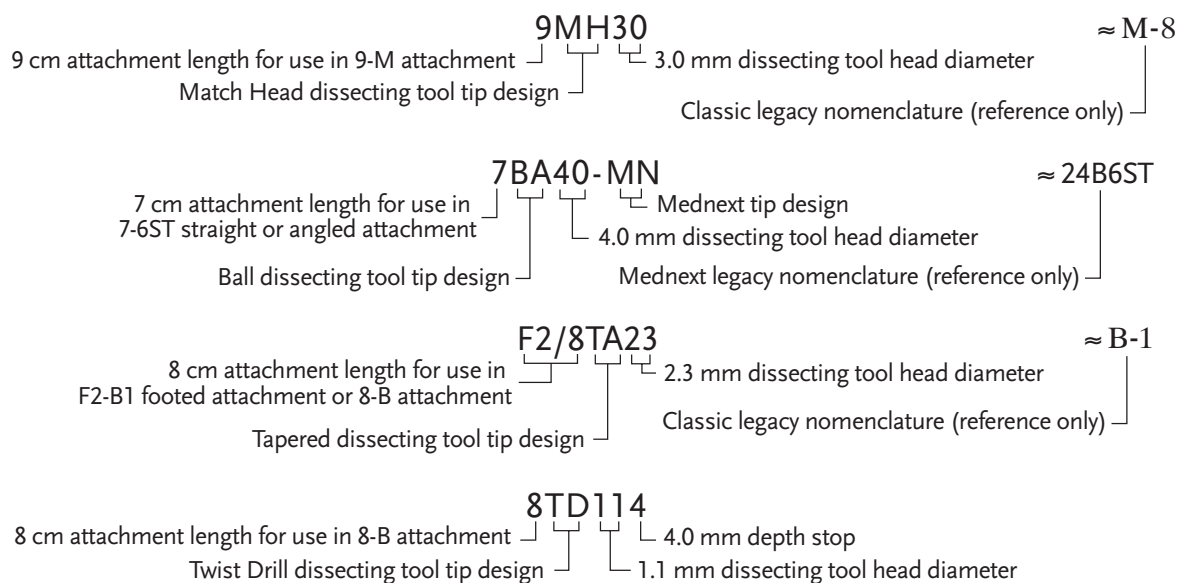

DISPOSABLE LEGEND ACCESSORIES *(continued)*

Additional Dissecting Tool Identification following Dissecting Tool Head Diameter

DIAMOND COATED DISSECTING TOOLS:

Available in:

- D Diamond, e.g., 15MH30D
- DC Diamond, coarse, e.g., 10BA50DC
- DX Diamond, extra coarse, e.g., 15CY65DX

BALL DISSECTING TOOLS:

Available in:

- L Long ball — elongated tool stem, e.g., 7BA20L-MN
- F Fine ball — fine fluted head, e.g., 7BA40F-MN
- C Carbide ball, e.g., 7BA40C-MN

TAPERED DISSECTING TOOLS:


Available in:

- S Spiral cutting flute, e.g., F2/8TA23S

CYLINDER DISSECTING TOOLS:

Available in:

- L Long cylinder — elongated cylinder head design, e.g., 14CY50L
- B Ball tip cylinder — rounded tip design on cylinder shape, e.g., 14CY50B




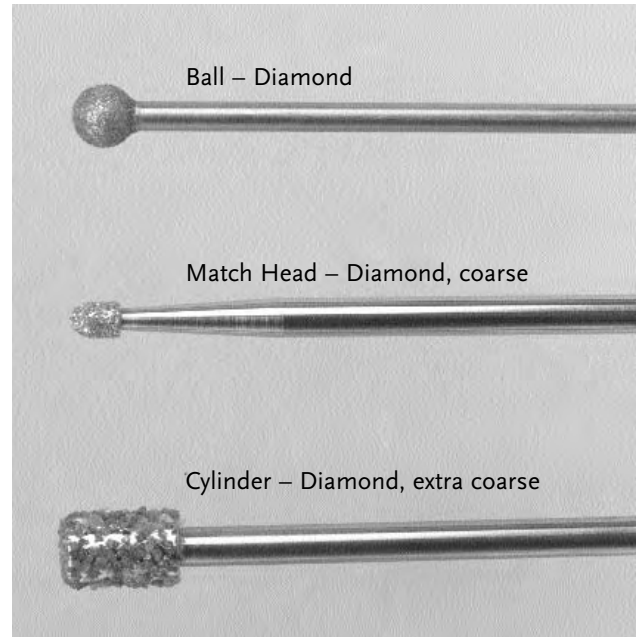


GENERAL GUIDELINES FOR ATTACHMENT AND TOOL APPLICATIONS





These are general guidelines for dissecting tool applications and are not an all-inclusive listing.

SURGERY	COMMONLY USED LEGEND ATTACHMENTS	COMMONLY USED TYPES OF LEGEND DISSECTING TOOLS	SUGGESTED LEGEND MOTOR(S)
---------	-------------------------------------	--	------------------------------

SPINE

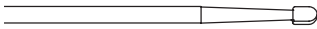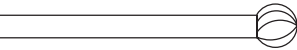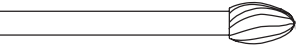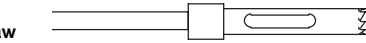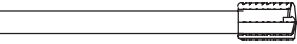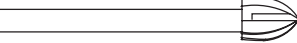8-B 9-M 14-AM straight or angled 15-A straight or angled	Match Head  Elongated spherical design allows controlled, delicate dissection. For laminotomy, foramenotomy, entry hole, nerve decompression, osteophyte removal...	Legend Gold Legend Gold Touch
	Ball  Helical cutting flutes dissect bone from a variety of approach angles. For debridement, decortication, corpectomy, osteophyte removal...	
	Oval  Helical cutting flutes and curved design blend acorn and ball styles to vary dissection efficiency with approach angle. For decortication, laminotomy, entry hole, nerve decompression, osteophyte removal...	
	Hole Maker/Saw  Matched sets of Hole Makers and Hole Saws are efficient and effective for interbody fusion...	
	Cylinder  Effective bone sculpturing and planing. For graft shaping, debridement, corpectomy, decortication, interbody fusion, fusion takedown...	
	Acorn  Curved designs varies dissection efficiency with varied approach angles. For entry hole, laminotomy, bone shaping, debridement, corpectomy, decortication, fusion takedown...	
Telescoping	Match Head  Elongated spherical design allows controlled, delicate dissection. For laminotomy, foramenotomy, entry hole, nerve decompression, osteophyte removal...	
Footed	Tapered  Slender design for precise dissection with minimal bone loss. For transection, osteotomy, graft harvesting, bone shaping, laminectomy...	

NEUROSURGICAL – CRANIAL

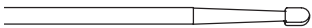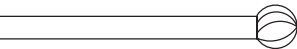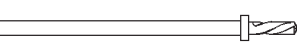
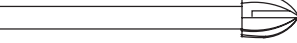
7-6ST straight or angled 8-B 9-M 10-9ST straight or angled 14-AM straight or angled 15-A straight or angled	Match Head  Elongated spherical design allows controlled, delicate dissection. For entry hole, nerve decompression, mastoidectomy, acoustic neuroma, skull base...	Legend Gold Legend Platinum
	Ball  Helical cutting flutes dissect bone from a variety of approach angles. For debridement, decortication, entry hole, nerve decompression, mastoidectomy, acoustic neuroma...	
	Twist Drill  Helical design with stop produces a hole with a precise depth. Ideal for plating...	
	Acorn  Curved design varies dissection efficiency with varied approach angles. For entry hole, bone shaping, debridement...	
Telescoping	Match Head  Elongated spherical design allows controlled, delicate dissection. For entry hole, nerve decompression, osteophyte removal, sinus dissection, mastoidectomy, acoustic neuroma, skull base...	
Footed	Tapered  Slender design for precise dissection with minimal bone loss. For craniotomy, transection, osteotomy...	

GENERAL GUIDELINES FOR ATTACHMENT AND TOOL APPLICATIONS *(continued)*


These are general guidelines for dissecting tool applications and are not an all-inclusive listing.

SURGERY	COMMONLY USED LEGEND ATTACHMENTS	COMMONLY USED TYPES OF LEGEND DISSECTING TOOLS	SUGGESTED LEGEND MOTOR(S)
---------	-------------------------------------	--	------------------------------





GENERAL SURGERY AND PLASTIC SURGERY (CRANIOFACIAL/MAXILLOFACIAL/STERNOTOMY)

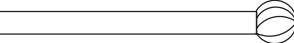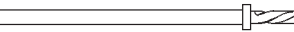7-6ST straight or angled 8-B 9-M 10-9ST straight or angled 14-AM straight or angled	Match Head  Elongated spherical design allows controlled, delicate dissection. For entry hole, nerve decompression, osteophyte removal, sinus dissection...	Legend Gold Legend Platinum
	Ball  Helical cutting flutes dissect bone or cement effectively from a wide variety of approach angles. For debridement, decortication, sinus dissection...	
	Tapered  Slender design for precise dissection with minimal bone loss. For transection, osteotomy, graft harvesting, bone shaping, entry hole, suture hole, midface advancement...	
	Twist Drill  Helical design with stop produces a hole with a precise depth. Ideal for plating...	


EAR, NOSE AND THROAT

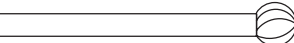7-6ST straight or angled 10-9ST straight or angled	Ball  Helical cutting flutes dissect bone or cement effectively from a wide variety of approach angles. For debridement, decortication, mastoidectomy, facial recess, cochlear implant bed, ossiculoplasty, labyrinthectomy, middle fossa approach, acoustic neuroma...	Legend Platinum
---	--	------------------------

ORTHOPAEDICS

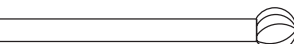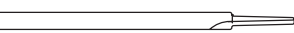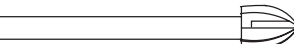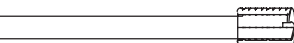8-B 9-M 14-AM straight or angled 21-TU 26-R Footed Telescoping	Ball  Helical cutting flutes dissect bone or cement effectively from a wide variety of approach angles. For debridement, decortication, cement removal, acetabular revision...	Legend Gold Legend Gold Touch
	Tapered  Slender design for precise dissection with minimal bone loss. For transection, osteotomy, graft harvesting, bone shaping, suture hole...	
	Acorn  Curved designs varies dissection efficiency with varied approach angles. For bone shaping, debridement, decortication, cement removal, acetabular revision...	
	Cylinder  Effective bone sculpturing and planing. For graft shaping, debridement, decortication, cement removal, acetabular revision...	
Footed	Tapered  Slender design for precise dissection with minimal bone loss. For transection, osteotomy, graft harvesting, bone shaping...	

BIOMETALS/BIOCERAMICS/BIOMATERIALS

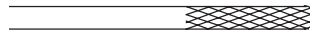MC	Metal Cutter  Cutting flutes or diamond wheel design remove metals, ceramics and other biomaterials effectively from a variety of approach angles. For cutting rods, pins, plates, implants, screws...	Legend Gold Legend Gold Touch Legend Platinum
-----------	---	--

POWER SOURCE REQUIREMENTS


Motors	Required Operating (Dynamic) Pressure	Nominal Operating (Dynamic) Pressure	Approximate Flow Rate Required	Gas Type
Legend Gold	80 – 120 psi	100 psi	12 cubic feet/min	Nitrogen or Dry-Filtered Compressed Air
	5.5 – 8.3 bar	6.9 bar	340 liters/min	
Legend Gold Touch	80 – 120 psi	100 psi	12 cubic feet/min	Nitrogen or Dry-Filtered Compressed Air
	5.5 – 8.3 bar	6.9 bar	340 liters/min	
Legend Platinum	80 – 120 psi	100 psi	12 cubic feet/min	Nitrogen or Dry-Filtered Compressed Air
	5.5 – 8.3 bar	6.9 bar	340 liters/min	

CAUTION

DO NOT run a Legend Motor at an operating pressure below or above the required operating pressure range. Operating pressure below 80 psi (5.5 bar) may not provide proper lubrication to the motor. Operating pressure above 120 psi (8.3 bar) may damage or reduce the life of the motor.


SYSTEM CONNECTIONS — Assembly for In-House Compressed Gas Source

In-house Gas Source with N₂ DISS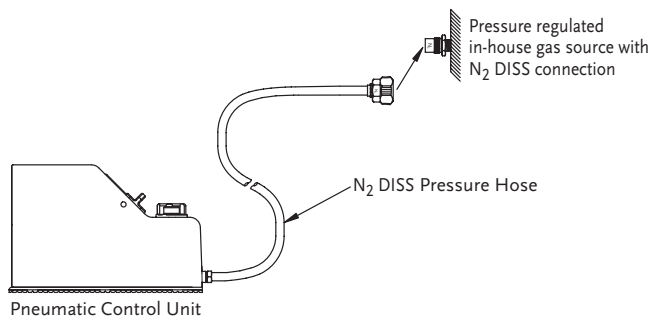


In-house Gas Source with N₂ DISS connects to N₂ DISS Pressure Hose on Pneumatic Control Unit.

In-house Gas Source with Female Schrader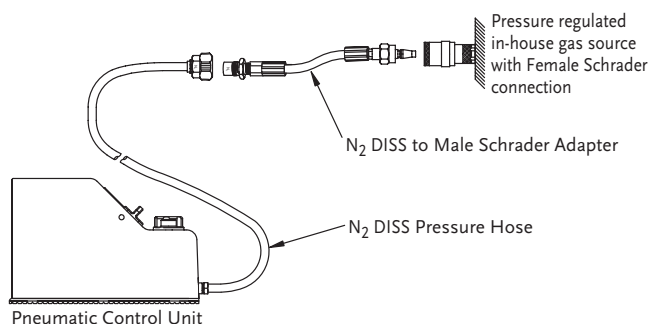

In-house Gas Source with Female Schrader connects to the N₂ DISS to Male Schrader Adapter. Connect Adapter to N₂ DISS Pressure Hose on Pneumatic Control Unit.

In-house Gas Source with Air DISS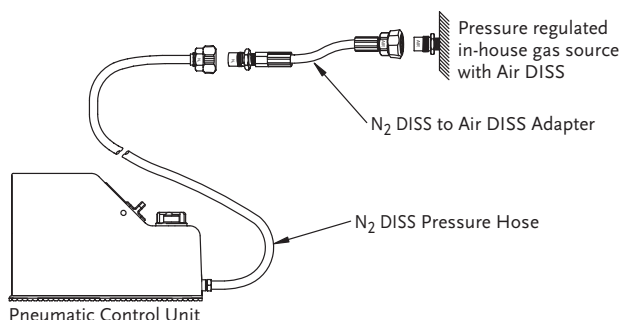


In-house Gas Source with Air DISS connects to the N₂ DISS to Air DISS Adapter. Connect Adapter to N₂ DISS Pressure Hose on Pneumatic Control Unit.

POWER SOURCE REQUIREMENTS *(continued)*

SYSTEM CONNECTIONS — Assembly for Cylinder Compressed Gas Source

Cylinder with Legend Regulator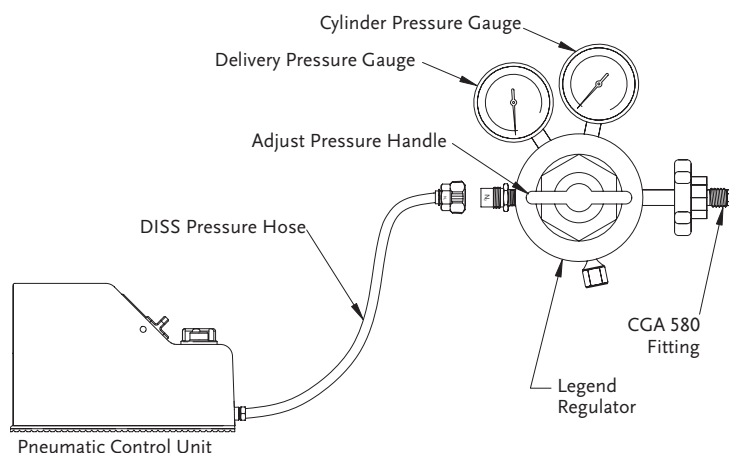


Cylinder with Legend Regulator connects to N₂ DISS Pressure Hose on Pneumatic Control Unit

Cylinder with Midas Rex Safety Valve Regulator

Step 1

Remove In-Line Oiler.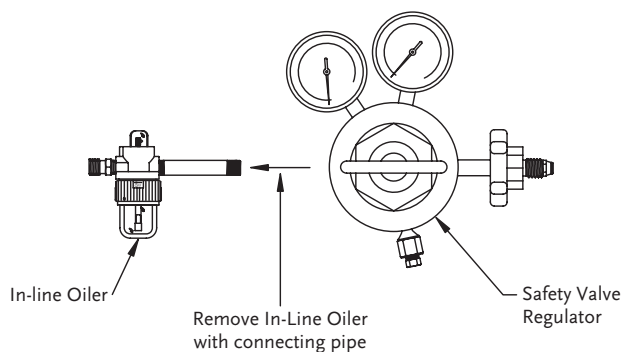


CAUTION

DO NOT use MIDAS REX In-line oiler system with the Legend System.

Step 2

- The N₂ DISS to WF4 Adapter connects to the Regulator.
- Connect the elbow adapter to the regulator, then connect the N₂ straight adapter to the elbow adapter.
- Connect Pneumatic Control Unit Pressure Hose on to the N₂ DISS Adapter.

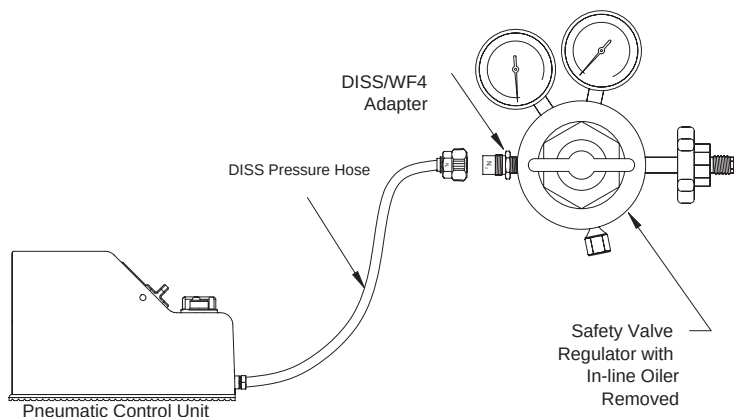1. Set Legend System non-running (static) pressure between 80–120 psi (5.5–8.3 bar) at compressed gas source. Operating (dynamic) pressure may be adjusted as needed when the motor has an attachment and dissecting tool installed and is running. (See Pre-operative Procedure #9)
2. a. Remove cover from Lubricant/Diffuser.
b. DO NOT remove inner foil seal of Lubricant/Diffuser.

3. Insert Lubricant/Diffuser into lubricant/diffuser housing located at base of motor hose.

4. Rotate Lubricant/Diffuser from “ ” position to “ ” position. It will come to a complete stop with a tactile click when it is fully seated and locked into the housing.

WARNING

Failure to properly secure Legend Lubricant/Diffuser may cause injury to operator and/or operating room staff.

NOTE

The lubrication rate is pre-set by the factory and, therefore, should not be altered.

CAUTION

- **DO NOT** use a Legend Motor without a lubricant/diffuser installed in the motor's lubricant/diffuser housing.
- **DO NOT** use a Legend Lubricant/Diffuser if it appears to be damaged or if inner foil seal is punctured.
- **DO NOT** exceed 1 hour drill time per surgery for a Legend Lubricant/Diffuser.
- **DO NOT** attempt to remove Legend Lubricant/Diffuser while the motor is in operation.
- **DO NOT** re-use a used Legend Lubricant/Diffuser.
② Legend Lubricant/Diffuser is a single use, disposable product.
- **DO NOT** refill a used Legend Lubricant/Diffuser.
- **DO NOT** steam or EO sterilize the Legend Lubricant/Diffuser. The lubricant/diffuser will melt into the lubricant/diffuser housing with potential damage to the motor. The lubricant/diffuser must be removed from the motor's lubricant/diffuser housing prior to sterilization.

Do not sterilize. Lubricant/diffuser will melt.

5. Connect the Legend Motor hose to the Legend port on the Legend Pneumatic Control Unit.

WARNING

DO NOT pinch, kink, obstruct or step on the motor hose. This may cause the motor hose to burst with potential injury to the patient, operator and / or operating room staff.

If using the Legend Gold Touch, slide the finger/foot control on the Legend Pneumatic Control Unit to the “” position. This will automatically depress and lock the foot pedal. The control will not lock into the “” position unless the motor is connected into the Legend port.

If using the TRITON® Power Surgical Instrumentation System in conjunction with the Legend Motor, connect the TRITON hose into the TRITON port on the Legend Pneumatic Control Unit.


WARNING

DO NOT use Legend Pneumatic Control Unit finger/foot control in the “” position to operate a Legend Gold or Legend Platinum Motor. The control is used in the “” position only for the Legend Gold Touch Motor.

The Legend Motor's exhaust hose may have an oily film on the external surface from pressure and/or temperature differentials following sterilization. Wipe exhaust hose with a sterile cloth prior to use. After cleaning, if motor continues to have oil on the exhaust hose, return the motor to MPSS for refurbish.

6. Prior to installation of a Legend Attachment and Dissecting Tool onto a Legend Motor, ensure that arrows on the motor collet flats are in proper alignment. If the arrows are not aligned, use the Legend motor wrench to turn the rotational collet (closest to the motor case) until its arrow is aligned with the arrow on the stationary collet (farthest away from the motor case.)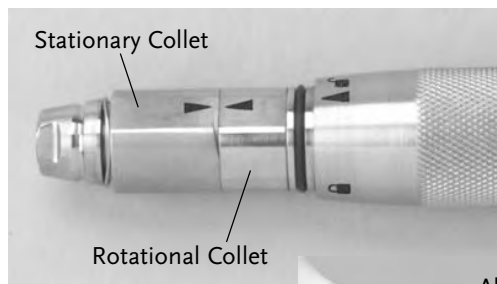

NOTE

A Legend Attachment will not seat onto a Legend Motor if the arrows on the collet flats are not in alignment.

DO NOT use any other components except for the Legend Motor Wrench to align arrows on motor collet.

Before installation of attachment and dissecting tool on the Legend Gold Touch Motor, ensure that the safety slide is in the “○” position.

Install a Legend Attachment and Dissecting Tool

Straight attachment installation is listed below. For more detailed information on all the various attachment and dissecting tool installation procedures, refer to the Attachment and Dissecting Tool Section.

7. Install a Legend Attachment and Dissecting Tool:

- a. Slide straight attachment over motor collet aligning triangular markers on attachment and motor case. A tactile and audible click will be observed when attachment is fully seated.

- b. Insert dissecting tool into attachment with a slight rotational motion. A tactile and audible click is observed indicating that the tool is fully seated.

CAUTION

Match the nomenclature and color code on the Legend Dissecting Tool packaging to the same nomenclature and color code on the Legend Attachment.

- c. Rotate attachment to the "🔒" position on the motor case. Gently pull on dissecting tool shaft to ensure proper installation.

CAUTION

Legend Motors should only be operated when the attachment is in the "🔒" position.

8. To activate the Legend Gold or Platinum Motors:
Depress the foot pedal on the Pneumatic Control Unit.
Allow motor to run for approximately 30 seconds. An increase in speed may be noticed as lubrication is provided to the motor.

WARNING

DO NOT use the Legend System if motor is leaking lubricant at motor collet/attachment.

DO NOT use the Legend System if dissecting tool is flailing.

DO NOT pinch, kink, obstruct or step on the motor hose. This may cause the motor hose to burst with potential injury to be patient, operator and / or operating room staff.

 To activate the Legend Gold Touch Motor:

- For the Legend Gold Touch, be certain the finger/foot control on the Legend Pneumatic Control Unit is locked in the “” position.
- With the Safety Slide in the “○” position, install and lock an attachment and dissecting tool on the motor.
- Lift the Finger Control and pull the Safety Slide to the “|” position.
- Depressing the Finger Control will now activate the motor. Allow motor to run for approximately 30 seconds. An increase in speed may be noticed as lubrication is provided to the motor.

WARNING

DO NOT use the Legend System if motor is leaking lubricant at motor collet/attachment.

DO NOT use the Legend System if dissecting tool is flailing.
See Troubleshooting section.

NOTE

 When using longer attachments with the Legend Gold Touch, the extension lever on the Finger Control may be extended to provide better digital control.

CAUTION

 When using angled attachments with the Legend Gold Touch, extend the lever on the Finger Control and hold by the attachment.

CAUTION

- If the motor has an uncomfortable temperature to touch/hold following sterilization, allow an adequate “cool down” period.
- If the motor has an uncomfortable temperature to touch/hold and an adequate “cool down” period was used, check that a new Legend Lubricant/Diffuser is properly attached to the motor’s lubricant/diffuser housing to help cool down motor. Run the motor briefly to assist in its cool down.

- Operating (dynamic) pressure may be checked diagnostically at the Legend Pneumatic Control Unit while the motor is running. Operating pressure will decrease slightly from the non-running (static) pressure setting when the motor is activated. Adjust operating pressure as needed at the compressed gas source until supply pressure gauge on Legend Pneumatic Control Unit reads within a range of 80 – 120 psi (5.5 – 8.3 bar) as required.


To ensure optimal motor performance, set nominal operating pressure to 100 psi (6.9 bar).

If the surgeon requests additional power during a procedure, compressed gas source may be turned up to an operating pressure of 120 psi (8.3 bar).


To decrease pressure, turn down the in-house compressed gas source or loosen the pressure handle on the regulator. Push down on the pressure relief at the pneumatic control unit to exhaust excess pressure in the hoses. Then re-adjust pressure as needed.

Legend Gold, Legend Gold Touch and Legend Platinum Lock

Legend Gold, Gold Touch and Platinum Motors should only be operated when the attachment is in the “” position on the motor.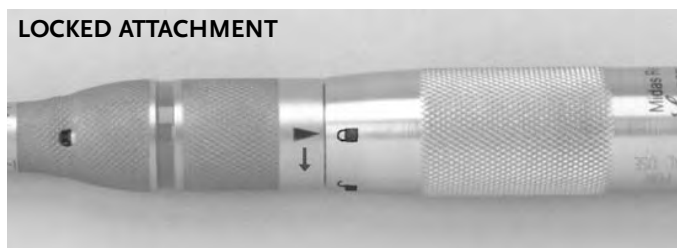


Motor will operate.

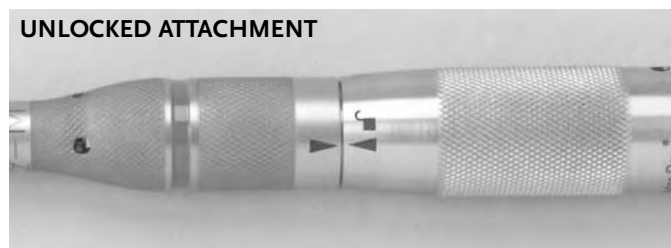
Motor will not operate.

Legend Gold Touch Lock

The Legend Gold Touch will operate with the safety slide in the “|” position and will not operate with the safety slide in the “○” position.

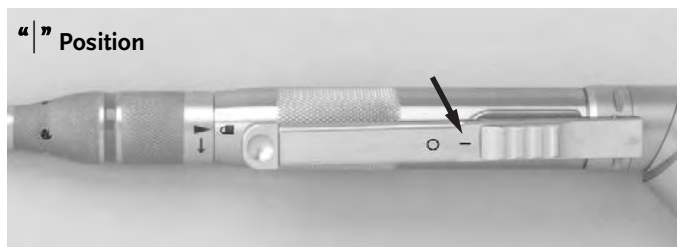Motor will operate.

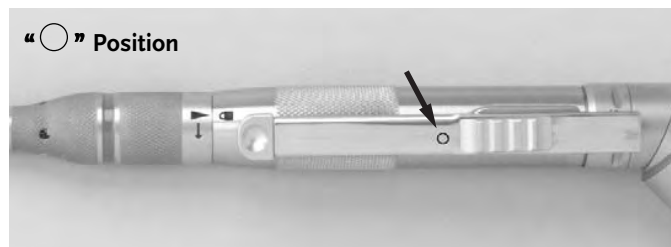Motor will not operate.

In addition, the Legend Gold Touch will not operate if the finger/foot control on the pneumatic control is in the “” position, unless the foot pedal is activated also.

Legend Gold Touch Motor will not operate.

DISASSEMBLY OF EQUIPMENT

1. Depressurize System.

Turn off the tank or wall outlet compressed gas source.

Release gas from the pressure hose by depressing the pressure relief on Legend Pneumatic Control Unit.

WARNING

DO NOT disassemble equipment before gas is released from Legend Pneumatic Control Unit and associated hoses.

DO NOT remove Legend Gold Touch Motor from Pneumatic Control Unit unless compressed gas has been released from the hoses.

2. Release Legend Motor Hose from Pneumatic Control Unit by holding the motor's exhaust hose firmly and depressing Legend port connection.

When releasing the Legend Gold Touch, depressing the Legend port connection will automatically release the “” position on the finger/foot control to the “” position. The foot pedal will move upward to its non-running position.

If using the **TRITON Power Surgical Instrumentation System**, hold the TRITON blue hose firmly and remove by turning TRITON quick connect coupler clockwise.

3. Unscrew Lubricant/Diffuser from housing. Discard Lubricant/Diffuser.

NOTE

During removal of lubricant/diffuser, oil residue may be present. An absorbent cloth should be used to facilitate removal.

CAUTION



DO NOT steam or EO sterilize the Legend Lubricant/Diffuser. The lubricant/diffuser will melt into the lubricant/diffuser housing with potential damage to the motor. The lubricant/diffuser must be removed from the motor's lubricant/diffuser housing prior to sterilization.

4. Remove attachment and discard used dissecting tools in an appropriate container.

Straight attachment removal is listed below. For more detailed information on the various attachment and dissecting tool removal procedures, refer to the Attachment and Dissecting Tool Section.

- To remove a straight attachment, hold motor in palm of hand. Rotate attachment to the “” position.
- Remove dissecting tool from the attachment and discard dissecting tool.
- Use thumb and index finger to lift attachment off motor.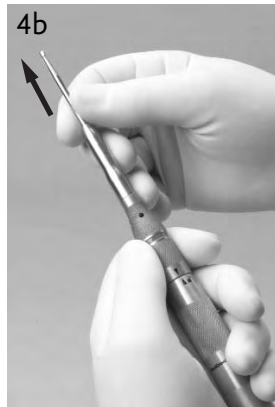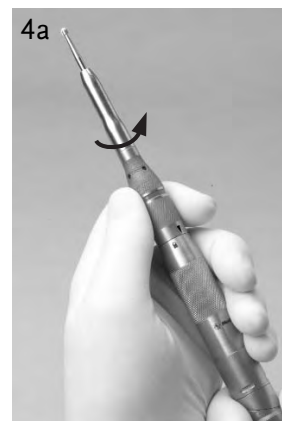

WARNING

DO NOT use or re-process a dissecting tool that has already been used. Using a dissecting tool that has already been used may cause injury to the patient, operator and/or operating room staff. Using a dissecting tool that has already been used may affect motor performance and will void manufacturer's limited warranty.

CLEANING

⚠ CAUTION

- **DO NOT** soak MEDTRONIC MIDAS REX LEGEND Equipment.
- **DO NOT** use ultrasound to clean MEDTRONIC MIDAS REX LEGEND Equipment.

Recommended Surgical Instrument Cleaning Solutions

Neutral enzymatic detergent, pH between 6.0–8.0.

⚠ CAUTION

- **DO NOT** use chlorine based or corrosive cleaning agents such as bleach, lye, etc.
- **DO NOT** use acetone.
- **DO NOT** use corrosive cleaning agents such as solutions containing glutaraldehyde, etc.

LEGEND MOTORS:

STEP 1:

Wipe all external surfaces of the motor and hose and wipe inner surface of lubricant/diffuser housing with a cloth dampened with surgical instrument cleaning solution.

STEP 2:

Brush motor case and collet with a nylon brush dampened with surgical instrument cleaning solution.



STEP 3:

Rinse motor thoroughly under running water, collet end pointed down. Dry collet and motor with lint free towel.

LEGEND ATTACHMENTS AND TELESCOPING TUBES:

STEP 1:

- Wipe all attachments and telescoping tubes with a cloth dampened with a surgical instrument cleaning solution. Attachments and tubes may be mechanically agitated in cleaning solution, but not soaked or immersed. A nylon brush dampened with a surgical instrument cleaning solution may be used to clean the external surfaces and internal connecting surfaces of the attachments and tubes. Move any moveable parts back and forth to allow solution to thoroughly clean attachment, e.g., sleeve on footed attachment, perforator attachment. Rinse thoroughly with tap water.
- Immerse head of Contra-Angle Attachment in surgical instrument cleaning solution and run motor for 1 minute.
- Straight attachments, footed attachments and telescoping straight tubes have special cleaning brushes sized to the attachment's or telescoping tube's internal diameter. Push the brush wet with surgical instrument cleaning solution through the attachment or telescoping tube from rear to front to loosen and remove debris trapped inside. Repeat as needed.

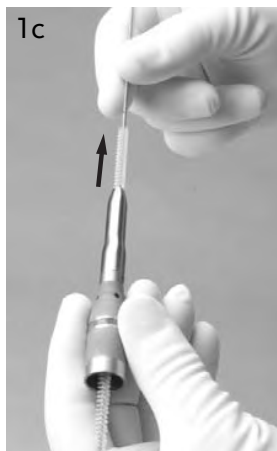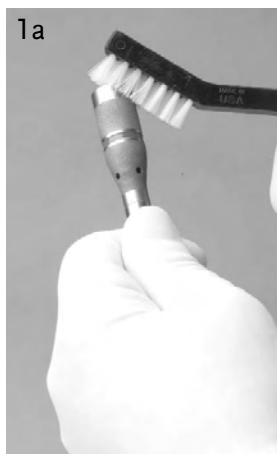CLEANING *(continued)*

NOTE

Cleaning brushes will not pass through angled attachments, contra-angle attachment, metal cutting attachment, perforator attachment or Jacobs chuck attachment because they are not cannulated.

- d. All attachments and telescoping tubes should be rinsed with tap water.

STEP 2:

Thoroughly dry attachments. An air gun may be used to blow moisture out from rear to front of attachment.

STEP 3:

The Legend Attachment Cleaning Nozzle attaches on to the recommended aerosol spray lubricant.

- a. Attach nozzle to aerosol spray can and push attachment onto nozzle by aligning arrows on attachment and cleaning nozzle.
- b. Rotate attachment slightly to ensure a tight fit.
- c. Cover attachment with paper towel.
- d. Spray in one 3-second squirt to remove debris and lubricate the attachment.
- e. Rotate attachment back to arrow on nozzle and pull attachment off of nozzle.
- f. Clean the nozzle for re-use.

STEP 4:

Prepare motor, attachments and telescoping tubes for sterilization.

LEGEND CLEANING NOZZLE FITS ON TO PANA SPRAY®

Recommend:

Pana Spray
NSK America Corp.
Telephone: 1-888-675-1675
<http://www.nsk-inc.com/products/catalog/accessories1.htm>

LEGEND PNEUMATIC CONTROL UNIT:

Cover motor ports with supplied plugs before cleaning. Wipe all external surfaces of the Pneumatic Control Unit and pressure hose with a cloth dampened with surgical instrument cleaning solution after each use.

CAUTION

If port plugs are not used, cleaning solutions could accumulate in the motor ports and may be propelled into the motor causing internal damages to the motor. Port plugs are attached to prevent debris ingress when port is not in use or during cleaning.

DO NOT immerse. **DO NOT** sterilize.

LEGEND INSTRUMENT CASE:

Wipe all external/internal surfaces of the Instrument Case with a cloth dampened with surgical instrument cleaning solution after each use.

WASHER-DISINFECTOR

This washer-disinfector guideline is only a recommended guideline and may vary among hospitals depending upon the type of washer-disinfector equipment and possible pathogens in their respective areas. This guideline represents exposure times and not total cycle time. The recommended washer-disinfector parameters may be modified based on standard hospital cleaning-disinfecting protocol.

Recommended Washer-Disinfector Cycle

Pre-Wash	Temperature:	35° C/95° F
	Time:	5 minutes
Main Wash	Temperature:	93° C/199° F
	Time:	30 minutes
Neutralize	Time:	2 minutes
Final Rinse	Temperature:	65° C/149° F
	Time:	10 minutes
POST CLEANING:		
• A cleaned device should be stored free of moisture. • Attachments and tubes must be relubricated as specified in the instruction manual. • Prior to surgical use a cleaned device must be sterilized according to the sterilization guidelines identified in the instruction manual.		
 CAUTION		
• The use of a washer-disinfector for cleaning may cause a pre-mature degradation in performance. • Remove devices from instrument case before placing into washer-disinfector. • Orient devices in the washer-disinfector by following manufacturer recommendations and allow device to drain. Lubricant / Diffuser Housing should be oriented to drain washer fluids.		

Recommended Cleaning Solutions

- Neutral enzymatic detergent, pH between 6.0-8.0.

CAUTION

DO NOT use chlorine based or corrosive cleaning agents on Medtronic Midas Rex re-usable devices or any other Midas Rex equipment.

STERILIZATION GUIDELINES FOR LEGEND MOTORS, ATTACHMENTS AND TELESCOPING TUBES

These sterilization guidelines are only recommended guidelines and may vary among hospitals depending upon the type of sterilization equipment and possible pathogens in their particular areas. These guidelines represent exposure time and not total cycle time. The recommended sterilization parameters may be modified based on standard hospital sterilization protocol.

Flash sterilization: Medtronic Powered Surgical Solutions does not recommend flash sterilization procedures. If flash sterilization is to be done, follow recommended guidelines of the Association of Operating Room Nurses (AORN) and/or the Association for the Advancement of Medical Instrumentation (AAMI).

Equipment	High Vacuum Steam (pre-vacuum) Temperature/Time	Gravity Displacement Temperature/Time	High Vacuum Steam (pre-vacuum) Temperature/Time	Ethylene Oxide (EO)
Legend Gold, Gold Touch and Platinum Motors Legend Attachments Legend Telescoping Tubes	270° F/132° C 4 minutes (wrapped/unwrapped) 8 minutes dry time	270° F/132° C 25 minutes (wrapped/unwrapped) 8 minutes dry time	273–278° F/134–137° C 3 minutes (wrapped/unwrapped) *consult reference below 8 minutes dry time	127–135° F/53–57° C 4 hours (wrapped) aeration 18 hours, 127–135° F/53–57° C

*Reference: For Items Contaminated With TSE Agents —

Devices may be decontaminated using steam sterilization at a temperature of 273 – 278° F / 134 – 137° C for a single cycle of 18 minutes or repeated for a total of six 3-minute cycles as referenced in NHS Estates HTM 2010 parts 4 & 6: Appendix 2, Items Contaminated With TSE Agents and WHO Infection Control Guidelines for Transmissible Spongiform Encephalopathies.

Transmissible Spongiform Encephalopathies (TSE): RETURN POLICY

Medtronic Powered Surgical Solutions (MPSS) will not authorize or accept the return of MPSS products that directly contact patients or is contaminated with a patient's body fluids who is suspected or confirmed with a Transmissible Spongiform Encephalopathies / Creutzfeldt-Jakob Disease (TSE/CJD) diagnosis. Furthermore, MPSS recommends that all Medtronic PSS products used on a patient confirmed with a TSE diagnosis be incinerated. Contact your Sales Representative for replacement of product incinerated under this policy or for temporary equipment while original equipment is quarantined. Contact MPSS Regulatory Affairs Department for additional information regarding TSE contamination. **MPSS dissecting tools used on a patient suspected of a TSE/CJD diagnosis must be incinerated.**

If TSE/CJD is excluded as a diagnosis, the quarantined reusable equipment may be returned for use after appropriate cleaning, decontamination and sterilization.

The following are recommended guidelines and may vary according to specific policy and procedures among hospitals. Hospital personnel should contact their infection control personnel for current procedures and policy for reusable equipment processing when suspect of contamination with Creutzfeldt-Jakob Disease (CJD) or other Transmissible Spongiform Encephalopathies (TSE).

Reusable Legend equipment that have been used on patients with suspected Creutzfeldt-Jakob Disease (CJD) or other Transmissible Spongiform Encephalopathies (TSE) should be quarantined and not reused until diagnosis is confirmed or excluded. Reusable Legend equipment should be quarantined after having been cleaned, decontaminated, sterilized and packed in a ridged sealed container until final diagnosis.

DO NOT immerse and/or soak reusable Legend equipment in any of the following solutions as irreversible corrosion or irreversible damage would occur.

Sodium hypochlorite/Bleach (NaOCL)

Caustic soda lye

Sodium hydroxide (NaOH)

Formic Acid (HCOOH)

WARNING

If a dissecting tool package is opened, but the tool is not used or contaminated, the tool can be re-sterilized by steam sterilization. Remove tool from all original packaging and place into an approved autoclave package. Steam sterilize as follows:

High Vacuum Steam

270° F (132° C) for 5 minutes

Gravity Displacement

270° F (132° C) for 15 minutes

The re-sterilized tool must be used promptly following re-sterilization. If rust or corrosion is encountered after re-sterilization, do not use the re-sterilized tool.

DO NOT use or re-process a dissecting tool that has already been used. Using a dissecting tool that has already been used may cause injury to the patient, operator and/or operating room staff. Using a dissecting tool that has already been used may affect motor performance and will void manufacturer's limited warranty.

CAUTION

DO NOT use low temperature hydrogen peroxide gas plasma sterilization due to the lumen internal diameter and length restrictions.

DO NOT use low temperature liquid peracetic acid sterilization due to immersion procedure.

DO NOT steam or EO sterilize the Legend Lubricant/Diffuser. The lubricant/diffuser will melt into the lubricant/diffuser housing with potential damage to the motor.

DO NOT steam or EO sterilize the Legend Attachment Cleaning Nozzle.

DO NOT steam or EO sterilize the Legend Pneumatic Control Unit.

Straight Attachments

To Install:

1. Slide straight attachment over motor collet aligning triangular markers on attachment and motor case. A tactile and audible click will be observed when attachment is fully seated.

NOTE

A Legend Attachment will not seat on a Legend Motor if the markers on the collet flats are not in alignment.

2. Insert dissecting tool into attachment with a slight rotational motion. A tactile and audible click is observed indicating that the tool is fully seated.

CAUTION

Match the nomenclature and color code on the Legend dissecting tool packaging to the same nomenclature and color code on the Legend attachment.

3. Rotate attachment to the “” position on the motor case. Gently pull on dissecting tool shaft to ensure proper installation.

NOTE

The Legend Motors will not run properly unless the attachment is in the “” position.

Smoke may be generated if attachment is not in the locked “” position.


To Remove:

1. Hold motor in palm of hand. Rotate attachment to the “” position.

2. Remove dissecting tool from the attachment and discard dissecting tool.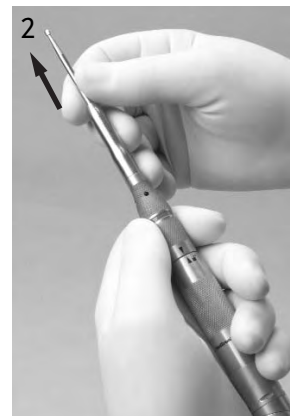

3. Use thumb and index finger to lift attachment off motor.

Angled Attachments

To Install:

1. Slide angled attachment over motor collet aligning triangular markers on attachment and motor case. A tactile and audible click will be observed when attachment is fully seated.

2. Rotate attachment to “” position on motor case.

3. With “Tool Lock” in the unlocked position, insert dissecting tool into angled attachment with a slight rotational motion. A tactile and audible click is observed indicating that the tool is fully seated.

4. Rotate Tool Lock to the “” position. Gently pull on dissecting tool shaft to ensure proper installation.


NOTE

A dissecting tool may be installed and locked in the attachment before the angled attachment is installed onto the motor.

Angled and straight attachments with the same length, marking and color band share the same dissecting tools.

The Legend Motors will not run properly unless the attachment is in the “” position.

Smoke may be generated if attachment is not in the locked “” position.

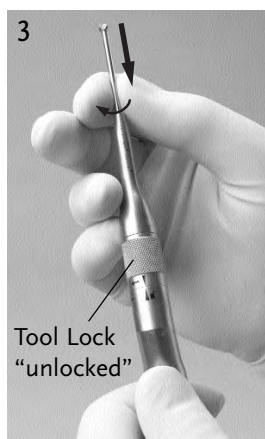To Remove:

Rotate the Tool Lock to the “” position to remove the dissecting tool from the attachment. Rotate the attachment to the “” position and lift attachment off of the motor.

CAUTION

When using an angled attachment, hold the handpiece assembly by the attachment so that the attachment does not inadvertently loosen from the handpiece.

Footed Attachments

Footed Attachments:	Use only the following Dissecting Tools:	Reference to Classic:
F1-B5	F1/8TA15 F1/8TA15S	B- 5 22S2CR (Mednext)
F2-B1	F2/8TA23 F2/8TA23S	B-1 B-1SP
F3-S	F3/9TA30	S-1

WARNING

DO NOT use a Legend Attachment if any part of the attachment appears to be bent, loose, missing or damaged. Excessive pressure or improper handling, such as bending or prying, with the attachment or dissecting tool may cause injury to patient, operator and/or operating room staff. If a Legend Attachment requires servicing or refurbishing, return attachment to MPSS.

To Install:

1. Insert dissecting tool into motor collet with a slight rotational motion. A tactile and audible click is observed indicating that the dissecting tool is fully seated.
2. Slide footed attachment over dissecting tool onto motor aligning triangular markers on attachment and motor case.

3. Pull footed attachment towards motor and rotate attachment to the "🔒" position on motor case.


NOTE

Legend footed attachments have a unique feature of actuating a distal moving sleeve to remove the attachment. This will prevent accidental disengagement of attachment from motor during use.

The Legend Motors will not run properly unless the attachment is in the "🔒" position.

To Remove:

1. To remove Legend footed attachment, hold motor in palm of hand. Push sleeve on footed attachment distally while rotating attachment to the "🔓" position on the motor case. Release sleeve.
2. To avoid injury from the dissecting tool, use thumb and index finger to cautiously and slowly lift attachment off of motor and away from the dissecting tool.
3. Pull dissecting tool out of motor collet and discard tool.

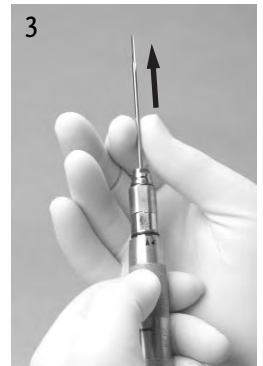WARNING

Remove Legend Footed Attachments cautiously and slowly to avoid injury to the operator.

Contra-Angle Attachment 16-MF

Dynamic Operating Pressure: 80 psi (5.5 bar)

CAUTION

DO NOT run the 16-MF attachment with operating pressure above 80 psi (5.5 bar). This may cause over-heating and damage to internal gears of attachment.

To Install:

1. Decrease pressure at the compressed gas source to 80 psi (5.5 bar).

Adjust pressure as needed by lowering pressure at gas source, then push down pressure relief on Pneumatic Control Unit to exhaust excess pressure in hoses. Verify at supply pressure.

2. Slide Contra-Angle Attachment over motor collet aligning triangular markers on attachment and motor case. A tactile and audible click will be observed when attachment is fully seated.


3. Rotate attachment to the “” position on the motor case.

4. Rotate attachment head's lever laterally to open position. Insert dissecting tool and return lever to closed position. Gently pull on dissecting tool shaft to ensure proper installation.

NOTE

A dissecting tool may be installed and locked in the attachment before the Contra-Angle attachment is installed on the motor.

The Legend Motors will not run properly unless the attachment is in the “” position.

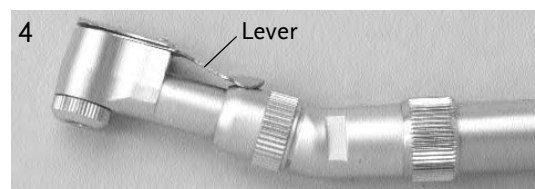Lateral View

Lever Open

Lever Closed

To Remove:

Rotate lever on attachment head laterally to remove the dissecting tool. Discard dissecting tool. Rotate the 16-MF attachment to the “” position and lift attachment off of the motor.

Metal Cutting Attachment

The Metal Cutting Attachment uses the tungsten carbide or diamond wheel dissecting tools. All metal cutting dissecting tools have an “MC” attachment prefix in their nomenclature, e.g., MC254, MC30.

Metal cutting dissecting tools cannot be installed into any other Legend Attachment.

WARNING

DO NOT use metal cutting dissecting tools on bone.

For metal transection, observe the following Safety Precautions:

- Eyewear protection is essential.
- Irrigate copiously to cool the cutting surfaces.
- Protect the wound site from metal debris.
- Use clamp or grasping device to control loose fragments during transection of any metal component.

To Install:

1. Slide Metal Cutting Attachment over motor collet aligning triangular markers on attachment and motor case. A tactile and audible click will be observed when attachment is fully seated.

Rotate attachment to the “” position on the motor case.

2. With tool lock unscrewed several turns and holding the attachment UPRIGHT, insert dissecting tool into attachment. Rotate dissecting tool until it drops into position and is fully seated. A tactile click is observed indicating that the tool is fully seated.

3. Firmly and completely tighten the tool lock. Pull on the tool shaft to ensure that it is properly installed.

NOTE

A dissecting tool may be installed and locked in the metal cutting attachment before the attachment is installed onto the motor.

The Legend Motor will not run properly unless the attachment is in the “” position.

To Remove:

1. Unscrew the tool lock with several turns, then withdraw the dissecting tool.
2. Rotate attachment to the “” position on the motor case and lift attachment off of the motor.

Telescoping Attachment

To Install:

1. Slide Telescoping Attachment over motor collet aligning triangular arrows on attachment and motor case. A tactile and audible click will be observed when attachment is fully seated.

Rotate attachment to the “” position on motor case.


2. Rotate TUBE Locking Ring towards “” by turning counterclockwise. Insert base end of selected telescoping tube into attachment. To lock tube in place, turn the TUBE Locking Ring clockwise until finger tight.

NOTE

DO NOT over tighten.

TUBE
Locking
Ring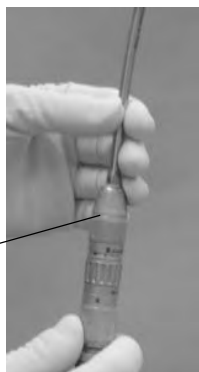

3. Be sure that the TOOL Locking Ring is in the “” position. Insert the dissecting tool into the telescoping tube. Tactile feedback is felt when the dissecting tool is fully seated. Turn the TOOL Locking Ring to the “” position. Verify that the tool is in place by gently pulling on the tool.

TOOL
Locking
Ring

4. If tube position needs to be changed, rotate TUBE Locking Ring towards “”, re-position tube, then rotate TUBE Locking Ring towards “”. Gently pull on dissecting tool, then the tube, to ensure proper installation.

NOTE

A dissecting tool may be installed and locked in the attachment before the telescoping attachment is installed on the motor.

Curved and straight telescoping tubes with the same length, marking and color band share the same dissecting tools.

Telescoping tubes are disposable following multiple uses and should be discarded when heat or excessive vibration is noticed, or when insertion of the tools becomes difficult.

The Legend Motor will not run properly unless the attachment is in the “” position.

To Remove:

1. Rotate the TUBE Locking Ring towards “”.
2. Rotate the TOOL Locking Ring to the “” position.
3. Pull telescoping tube out of attachment.
4. Rotate the attachment to the “” position on the motor case and lift attachment off of motor.

WARNING

Dissecting tool flutes are sharp and may perforate surgical gloves. Dissecting tools for the Telescoping Attachment may be grasped with a hemostat to aid in installation and removal.

Perforator Attachment

The Perforator Attachment has a Hudson chuck to drive any device with a Hudson shank, i.e., cranial perforator device.

To Install:

1. Slide Perforator Attachment over motor collet aligning triangular markers on attachment and motor case. A tactile and audible click will be observed when Perforator Attachment is fully seated.

2. Rotate Perforator Attachment to the “” position on the motor case.

3. To install a cranial perforator device with a Hudson shank, pull back proximally on the collar of the Perforator Attachment. Insert device and release collar to its original position.

NOTE

Cranial perforator device may be installed in the attachment before the perforator attachment is installed on the motor.

MPSS does not provide cranial perforator devices.

The Legend Motors will not run properly unless the attachment is in the “” position.

To Remove:

Pull back proximally on the collar of the Perforator Attachment to remove cranial perforator device. Rotate the Perforator Attachment to the “” position and lift attachment off of the motor.

WARNING

CONSULT the cranial perforator device labeling for the recommended speed specifications.

Maximum Speed of Perforator Attachments with Legend Pneumatic and EHS Motors

Pneumatic Motors		
Gas Pressure (Dynamic)	AD01 Output Speed (Max.)	AD03 Output Speed (Max.)
80 psi	660 rpm	850 rpm
100 psi	820 rpm	1050 rpm
120 psi	890 rpm	1,140 rpm
EHS Motor		
EHS Console Setting	AD01 Output Speed (Max.)	AD03 Output Speed (Max.)
60,000 rpm	645 rpm	830 rpm
70,000 rpm	745 rpm	965 rpm
72,000 rpm	770 rpm	995 rpm
74,000 rpm	790 rpm	1020 rpm
75,000 rpm	805 rpm	1035 rpm

Jacobs® Chuck Attachment

The Jacobs Chuck Attachment is a non-cannulated 5/32" chuck with key for drilling.

To Install:

1. Slide Jacobs Chuck Attachment over motor collet aligning triangular markers on attachment and motor case. A tactile and audible click will be observed when Jacobs Chuck Attachment is fully seated.

2. Rotate Jacobs Chuck Attachment to the "🔒" position on the motor case.

3. To install a drill bit, turn Jacobs key to open ridged collar. Insert drill bit and tighten collar with key.

NOTE

A drill bit may be installed in the attachment before the Jacobs Chuck attachment is installed on the motor.

MPSS does not provide drill bits.

The Legend Motors will not run properly unless the attachment is in the "🔒" position.

To Remove:

Use Jacobs key to open collar. Remove and discard drill bit in an appropriate container. Rotate the Jacobs Chuck Attachment to the "🔓" position and lift attachment off of the motor.

Contact your Medtronic Neurologic Technologies sales representative, or call:

Medtronic Powered Surgical Solutions Technical Services : 8 a.m. to 6 p.m. (Central Standard Time)
Telephone: 800-643-2773 or 817-788-6400

Outside the U.S.A., contact your nearest Medtronic Regional Distributor or Medtronic Neurological Sales Representative or call U.S.A. Telephone 817-788-6400

NO LATEX POLICY:

MPSS verifies that all Legend equipment and products identified in this manual manufactured by and/or for MPSS do not contain latex.

MPSS packaging materials, labels, pouches, package inserts or similar items do not contain latex.

Material Safety Data Sheet:

Material Safety Data Sheet on lubricant used in Legend Lubricant/Diffuser is available by contacting Regulatory Affairs Department or Technical Service at telephone numbers listed above.

LEGEND SYSTEM TROUBLESHOOTING

If Legend components require servicing or refurbishing, return components to MPSS for quality assured service by factory-trained personnel who will utilize genuine MEDTRONIC MIDAS REX LEGEND parts as required.

All Legend System components returned for servicing or refurbishing should be properly cleaned and sterilized prior to shipping.

PROBLEM	POSSIBLE CAUSE	RECOMMENDATIONS
LEGEND MOTORS		
Motor does not run or is low on power.	Hoses not properly connected. Operating pressure inadequate. Attachment not properly installed and locked onto the motor. (possible smoking condition) Motor sterilized with single use, disposable Lubricant / Diffuser cartridge attached to the motors lubricant/diffuser housing. Foot Pedal on Pneumatic Control Unit not properly functioning. Motor stall or will not spin.	Check all connections from compressed gas source to motor Check gas supply to Pneumatic Control Unit at Supply Pressure gauge. Pressure may need to be increased according to compressed gas requirements. Remove and re-install attachment and dissecting tool to ensure proper installation and locking of attachment onto motor. DO NOT use. Return motor to MPSS to be refurbished. Check for obstruction under foot pedal. If foot pedal continues not to function properly, return the Pneumatic Control Unit to MPSS to be refurbished. Manually spin the dissecting tool then activate the motor. If motor continues to stall, then return motor to MPSS to be refurbished.

LEGEND SYSTEM TROUBLESHOOTING *(continued)*

PROBLEM	POSSIBLE CAUSE	RECOMMENDATIONS
LEGEND MOTORS <i>(continued)</i>		
 Legend Gold Touch will not run after checking possible causes listed previously for all Legend motors.	Finger/Foot control on the Pneumatic Control Unit not activated. Safety slide in “○” position.	Push finger/foot control on Pneumatic Control Unit to the “  ” position. Move safety slide to “ ” position.
Motor continues to run.	Pneumatic Control Unit is not properly functioning. Finger control is not properly functioning.	Depressurize system. Return Pneumatic Control Unit to MPSS to be refurbished. Return motor to MPSS to be refurbished.
Motor is missing a component or a component has detached from the motor.	Motor failed.	DO NOT use. Return motor to MPSS to be refurbished.
Motor rotates, but an abnormal noise is heard.	Lubrication inadequate. Motor's exhaust hose damaged or cut, or internal pressure hose detached. Bearings of motor or attachment worn. Attachment not in proper locked position. (possible smoking condition)	Check for proper installation of the Legend Lubricant/Diffuser. If problem persists, replace Legend Lubricant/Diffuser. If problem continues, return motor to MPSS to be refurbished. DO NOT use. Depressurize system. Return motor to MPSS to be refurbished. Change attachment to isolate location of problem to the attachment or motor. Return motor and/or attachment to MPSS to be refurbished. See instructions for proper locking of attachments.
Motor/collet displays excessive rust.	Incorrect cleaning or sterilization method. Use of chlorine based or corrosive cleaning agents. Aging	Return to MPSS for refurbishing/purchase.

LEGEND SYSTEM TROUBLESHOOTING *(continued)*

PROBLEM	POSSIBLE CAUSE	RECOMMENDATIONS
LEGEND MOTORS <i>(continued)</i>		
Motor is heating to an uncomfortable temperature to touch/hold.	Inadequate “cool down” period following sterilization. Lubrication inadequate. Attachment transferring heat to motor. Heavy side loading during dissection. Inadequate irrigation	Motor must be allowed to cool down following steam sterilization. Run motor for 2 minutes. If motor continues to overheat, return the motor to MPSS to be refurbished. Check for proper installation of a new Legend Lubricant/Diffuser into motor’s lubricant/diffuser housing. Change attachment to isolate location of heating to the attachment or the motor. If attachment is source of heat, return attachment to MPSS to be refurbished. If motor is source of heat, return the motor to MPSS to be refurbished. Discontinue use and rest the motor by using intermittently or wrap the motor interface with a moist sterile towel. If motor continues to overheat, return the motor to MPSS to be refurbished. Ensure adequate irrigation to surgical site during bone dissection.
Motor is leaking lubricant.	Motor failed.	DO NOT use. Return motor to MPSS to be refurbished. If lubricant entered surgical site, treat patient per physician’s discretion.
Oily film on green exhaust hose of motor following sterilization.	Oily film on hose due to pressure and/or temperature differentials following sterilization.	Wipe the external surface of the motor hose with a sterile cloth prior to use. After wiping with sterile cloth, if motor continues to have oil on the hose, DO NOT use. Return the motor to MPSS to be refurbished.
Hose explodes.	Hose pinch, kink, obstruct or step on.	DO NOT use. Return motor to MPSS to be refurbished / purchase.

LEGEND SYSTEM TROUBLESHOOTING *(continued)*

PROBLEM	POSSIBLE CAUSE	RECOMMENDATIONS
LEGEND ATTACHMENTS or TELESCOPING TUBES		
Attachment or Telescoping Tube has uncomfortable temperature to touch/hold.	Heat from worn attachment/tube bearings. Attachment/tube unclean due to improper cleaning procedures. Heavy side loading during dissection.	DO NOT use. Try another attachment/tube. Return over-heated attachment to MPSS to be refurbished. Telescoping Tubes are multi-use disposable. If problem is resolved with a new Telescoping Tube, discard the over-heated tube. Check that appropriate cleaning procedures are being followed. Discontinue use and rest the attachment by using intermittently, try another identical attachment or wrap the attachment interface with a moist sterile towel. If attachment continues to overheat, return the attachment to MPSS to be refurbished/ purchase.
Color band on Attachments/Telescoping Tubes fades or discolors.	Incorrect cleaning or sterilization method. Use of chlorine based or corrosive cleaning agents. Aging	Use nomenclature markings on the attachment to match with a corresponding dissecting tool or return to MPSS for refurbishing/ purchase. Telescoping Tubes are multi-use disposable.
Attachment/Telescoping Tube displays excessive rust.	Incorrect cleaning or sterilization method. Use of chlorine based or corrosive cleaning agents. Aging	DO NOT use. Return attachment to MPSS to be refurbished/purchase. Dispose of telescoping tube. Telescoping Tubes are multi-use disposable.
Attachment/Telescoping Tube is bent, loose, damaged or missing a component.	Attachment mishandled, failed due to extended use or excessive force applied during use.	DO NOT use. Return attachment to MPSS to be refurbished/purchase. Dispose of telescoping tube. Telescoping Tubes are multi-use disposable.

LEGEND SYSTEM TROUBLESHOOTING *(continued)*

PROBLEM	POSSIBLE CAUSE	RECOMMENDATIONS
LEGEND ATTACHMENTS or TELESCOPING TUBES <i>(continued)</i>		
Attachment will not properly seat on motor.	Motor collet flats are not in proper alignment.	Use Legend Motor Wrench to rotate the flat closest to the motor case until its marker is aligned with the marker on the flat farthest away from the motor case.
Attachment has excess lubrication.	Over lubrication during cleaning process.	Visually inspect and wipe excess lubrication.
Difficulty removing tool from attachment.	Aging of attachment. Improper cleaning. Use of re-processed tools. Use of unauthorized re-furbisher.	Return to MPSS for refurbishing/purchase.
Footed attachment has a component missing from leg/foot area.	Attachment damaged by dissecting tool drilling out part or all of leg/foot area.	DO NOT use. Return attachment and motor to MPSS to be refurbished/purchase.
Overheating of 16-MF Contra-Angle Attachment.	Contra-Angle attachment operates by a set of internal gears to engage the drive shaft. It is normal for some heat to be generated approximately 2 cm from the distal end of the attachment and at the right angle head.	Verify pressure setting is at 80 psi (5.5 bar). DO NOT use if heat continues or is excessive. Return attachment to MPSS to be refurbished/purchase.
Perforator running too slow.	Pressure set incorrectly.	Check pressure setting at foot control.
LEGEND DISSECTING TOOLS		
Dissecting Tool flails.	Device other than a Legend Dissecting Tool used. Worn attachment/tube bearings. Attachment/tube and tool combination improper.	DO NOT use. Replace with a Legend Dissecting Tool. DO NOT use. Try another attachment/tube to isolate location of problem. If flail is produced by a specific attachment, return attachment to MPSS to be refurbished/purchase. Dispose of telescoping tube. Telescoping Tubes are multi-use disposable. DO NOT use. Check for the correct nomenclature and color code on the dissecting tool packaging to the same nomenclature and color code on the attachment/tube.

LEGEND SYSTEM TROUBLESHOOTING *(continued)*

PROBLEM	POSSIBLE CAUSE	RECOMMENDATIONS
LEGEND DISSECTING TOOLS <i>(continued)</i>		
Dissecting Tool flails. <i>(continued)</i>	If proper attachment/tube and tool combination is being used, motor may be damaged. Dissecting tool's size and geometry may contribute to flailing at certain speeds.	DO NOT use. Return motor to MPSS to be refurbished. Increase or decrease speed by changing pressure setting or foot/ finger control. DO NOT use if flailing persists. Change to a different dissecting tool.
Excessive vibration.	Dissecting tool's size and geometry may create excessive vibration at certain speeds.	Increase or decrease speed by changing pressure setting or foot/finger control. Change to a different dissecting tool.
Dissecting tool sterile packaging damaged. Tool outside the tube or tube opened.	Shipping damage - Dissecting tool is no longer sterile.	DO NOT use, contact MPSS Customer Service or Sales representative.
Dissecting tool sterile packaging opened, tool not used or contaminated.	Dissecting Tool not used during surgical procedure - Dissecting tool is no longer sterile.	See instruction for re-processing tools under sterilizations guideline section.
Expired Shelf Life.	Old product - Dissecting tool may no longer be sterile.	DO NOT use, contact MPSS Customer Service or Sales representative.
Dissecting tool will not seat properly in motor collet or angled attachment collet.	Debris in collet of angled attachment or motor. Device other than Legend Dissecting Tool being used.	Clean angled attachment or motor thoroughly as per instructions. If cleaning does not correct the problem, DO NOT use. Return angled attachment or motor to MPSS to be refurbished. DO NOT use. Replace with a Legend Dissecting Tool.
Dissecting tool rusts.	Incorrect storage. Reprocessed unused dissecting tool.	DO NOT use; return to MPSS. DO NOT use.
Dissecting tool fractures or breaks.	Excessive pressure. Reprocessed tool. Tool failure.	Remove all tool debris from patient. Check if reprocessed tool was used. Contact MPSS Customer Service or Sales Representative.
Dissecting tool dull.	Extended use. Reprocessed tool. Incorrect geometry.	Change to a new tool. Check if reprocessed tool was used. Contact MPSS Customer Service or Sales Representative.

REFURBISHING OR REPAIRS

When the Midas Rex Legend Pneumatic High Speed System requires servicing or refurbishing, contact Medtronic Powered Surgical Solutions Repair Services for a return authorization and instructions for returning the equipment to Medtronic Powered Surgical Solutions, Fort Worth, Texas. Medtronic Powered Surgical Solutions provides quality assured service by factory-trained personnel who will utilize genuine Midas Rex Legend parts as required.

Contact:

Medtronic Powered Surgical Solutions Repair Services:

(800) 335-9557 or (817) 788-6440

mmrcustomerservice@medtronic.com

Peak performance, reliability and maximum service life from your Midas Rex Legend System may be assured by using only those Midas Rex Legend products for your Midas Rex Legend System that are manufactured by and sold through Medtronic Powered Surgical Solutions, Fort Worth, Texas. While Medtronic Powered Surgical Solutions guarantees complete compatibility among its products within a specific product line, the dissecting tools are designed for single-use only, and Medtronic disclaims any responsibility when reprocessed dissecting tools are used. If you would like more information about the patient and product risks associated with reprocessed tools, please contact the number or e-mail address listed above.

Due to safety and environmental concerns, Medtronic Powered Surgical Solutions requests the return of pneumatic high speed motors for proper disposal at the end of the product life cycles.

LEGEND PNEUMATIC HIGH SPEED PREVENTIVE MAINTENANCE/SERVICE PROGRAM

The Midas Rex Legend Pneumatic High Speed System Preventive Maintenance Manual has been developed to assist you in getting the greatest ownership value from your Midas Rex Legend Pneumatic High Speed System, while helping to maximize its performance, safety and reliability. The scheduled preventive maintenance/service program is in addition to the required routine cleaning after each use. Please refer to the preventive maintenance manual for the specific steps necessary to maintain Midas Rex Legend Pneumatic High Speed System.


MANUFACTURER'S RECOMMENDATION

It is strongly recommended that surgeons, operating room personnel, and service personnel become thoroughly familiar with all Midas Rex Legend Systems prior to use in surgery. To assist in this effort, Medtronic Powered Surgical Solutions offers several Hands-On Workshops. Contact your Medtronic Neurologic Technologies Sales Representative or your Medtronic Regional Distributor for schedules.

MIDAS REX LEGEND PNEUMATIC HIGH SPEED SYSTEM LIMITED WARRANTY* (U.S. CUSTOMERS ONLY)

- A. This Limited Warranty provides the following assurance to the purchaser of a Medtronic Midas Rex Legend Pneumatic High Speed System. The Midas Rex Legend Pneumatic High Speed System includes the motor, foot control, instrumentation cases and trays (hereafter referred to as Motor Components), the attachments, dissecting tools and other accessories not listed above and telescoping tubes (hereafter referred to as Accessory Components) and jointly referred to as Legend Pneumatic High Speed System, unless specifically noted.
- (1) Should a Motor Component fail to function within normal tolerances due to a defect in materials or workmanship within a period of one year from the date of shipment of the new Motor Component, or 180 days from the date of shipment of the serviced or refurbished Motor Component to the purchaser (but not less than the one year period from the date of shipment of the new Motor Component), or
 - (2) Should an Accessory Component fail to function within normal tolerances due to a defect in materials or workmanship within a period of 90 days from the date of shipment of a new Accessory Component or serviced or refurbished Accessory Component to the purchaser, Medtronic will at its option: (a) issue a credit to the purchaser of the replacement Motor Component or Accessory Component equal to the Purchase Price, as defined in Subsection A(3), against the purchase of any same Motor Component or Accessory Component requested as its replacement, or, (b) provide a functionally comparable replacement Motor Component or Accessory Component at no charge.
 - (3) As used herein, Purchase Price shall mean the lesser of the net invoiced price of the original or current functionally comparable, or replacement Motor Component or Accessory Component.
- B. To qualify for this Limited Warranty, these conditions must be met:
- (1) The Legend Pneumatic High Speed System must be used in accordance with the labeling and Instruction Manual provided with the products.
 - (2) Only Legend Pneumatic High Speed System dissecting tools that are manufactured, authorized, and quality controlled by Medtronic may be used.
 - (3) No reprocessed dissecting tools may be used.
 - (4) The Legend Pneumatic High Speed System may not be damaged by accident, abuse, misuse or tampering.
 - (5) The Legend Pneumatic High Speed System may not be modified without the written permission of Medtronic.
- C. This Limited Warranty is limited to its express terms. In particular:
- (1) Except as expressly provided by this Limited Warranty, MEDTRONIC IS NOT RESPONSIBLE FOR ANY DIRECT, INCIDENTAL OR CONSEQUENTIAL DAMAGES BASED ON ANY DEFECT, FAILURE OR MALFUNCTION OF THE LEGEND PNEUMATIC HIGH SPEED SYSTEM TO FUNCTION WITHIN NORMAL TOLERANCES WHETHER THE CLAIM IS BASED ON WARRANTY, CONTRACT, NEGLIGENCE, STRICT LIABILITY, OR OTHER TORT OR OTHERWISE.
 - (2) This Limited Warranty is made only to the purchaser. AS TO ALL OTHERS, MEDTRONIC MAKES NO WARRANTY, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, WHETHER ARISING FROM STATUTE, COMMON LAW, CUSTOM OR OTHERWISE.
 - (3) The exclusions and limitations set out above are not intended to, and should not be construed so as to contravene mandatory provisions of applicable law. If any part or term of this Limited Warranty is held to be illegal, unenforceable or in conflict with applicable law by a court of competent jurisdiction, the validity of the remaining portions of the Limited Warranty shall not be affected, and all rights and obligations shall be construed and enforced as if this Limited Warranty did not contain the particular part or term held to be invalid. This Limited Warranty gives the purchaser specific legal rights. The purchaser may also have other rights that vary from state to state.
 - (4) No such express or implied warranty to the purchaser shall extend beyond the period specified in A(1) above.
 - (5) This Limited Warranty shall be the exclusive remedy available to any person.
 - (6) No person has any authority to bind Medtronic to any representation, condition, or warranty, except this Limited Warranty.

*This Limited Warranty is provided by Medtronic Powered Surgical Solutions, 4620 North Beach Street, Fort Worth, Texas 76137-4116. It applies only in the United States. Areas outside the United States should contact their local Medtronic representative for terms of the warranty.

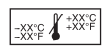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Jacobs is a registered trademark
Pana is a trademark of
All rights reserved.
Medtronic B.V.
of Powered Tool Holders, Inc.
NSK America Corporation